# Dermatix<sup>®</sup> Silicone Sheet

WITH SILON<sup>®</sup> TECHNOLOGY

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## AVAILABLE IN TWO FORMS:

**DERMATIX<sup>®</sup> SILICONE SHEET CLEAR** Virtually invisible!  
Apply on exposed areas. Perfect for face & hands – can even apply make-up over it.

**DERMATIX<sup>®</sup> SILICONE SHEET FABRIC** With a silky fabric backing for optimal comfort and convenience under clothing.

## INSTRUCTIONS FOR USE

**DESCRIPTION:** Dermatrix<sup>®</sup> Silicone Sheet Clear and Dermatrix<sup>®</sup> Silicone Sheet Fabric are sheets of adhesive silicone used to reduce or prevent hypertrophic scars and keloids.

**USE:** Consistent use of Dermatrix<sup>®</sup> Silicone Sheet Clear or Dermatrix<sup>®</sup> Silicone Sheet Fabric can minimize existing hypertrophic scars and keloids or it may be used to aid in the prevention of hypertrophic scars and keloids.

### INSTRUCTIONS:

**STEP 1:** Remove the product from its package. If necessary, use scissors to trim the product to size. It should extend slightly beyond the area of the scar.

**STEP 2:** Peel away the paper protective liner and save it for later use. Place the product "sticky side" down, directly on the scarred area. Keep the product in place for at least 12 hours and up to 23 hours per day.

**STEP 3:** At least once a day, wash the scarred area and both sides of the product with mild soap and water. Do not use a soap that contains moisturizers, lotions, or oils. A plain basic soap is recommended. Rinse the product thoroughly to remove all traces of soap. Allow the product to air dry or pat it dry with a lint free towel before reapplying. Between uses, store the product on the paper protective liner.

**SPECIAL RECOMMENDATIONS AND PRECAUTIONS:** Do not use creams, lotions or ointments with this product as they may interfere with the performance of the product and/or cause skin irritation.

**CAUTION:** As with any topical material applied directly to the skin's surface, some type of reaction (usually a sweat rash) is possible. Stop using the product if you have a reaction. After your skin has returned to normal, you may reapply the product for short periods and gradually increase the hours of usage. Monitor the scarred area for signs of reaction. Continue to increase the wearing time until a comfortable maximum has been reached. If you continue to have a problem, stop use and consult your physician.

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Silon is a registered trademark of Bio Med Sciences, Inc. © All rights reserved. This product is subject to US & Foreign Patents including US #s 5,656,279, 5,980,923 & 7,087,135. REV 01/06/09
Meda Pharmaceuticals Ltd, Skyway House, Parsonage Road, Takeley, Bishop's Stortford, CM22 6PU, UK